
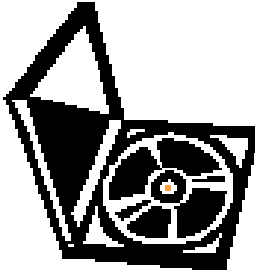
**FOGLIO D'USO
MODE D'EMPLOI
INSTRUCCIONES PARA EL USO
OPERATIONAL SHEET
ANWENDUNGSBLATT**

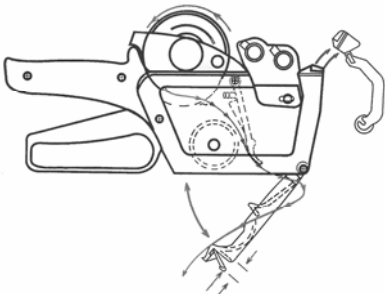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ENGLISH - HOW TO USE THE TRACKING SYSTEM

- A. Prior to each cycle, fill the relevant CYCLE FORM
- B. Insert the label roll in the labelling machine,.
- C. Set the machine by entering the batch number, packing and expiry date. USER OR AUTOCLAVE CODE (if there is more than one autoclave in the surgery) Proceed to print the labels, by means of the appropriate lever on the labelling machine
- D. The first printed label must be put onto the CYCLE FORM. At the same time, the same number of labels as bags to be put into the autoclave must be printed.
- E. Stick the labels directly onto the paper side of the bag
- F. Put then the bags into the autoclave and start the sterilization cycle. When the cycle has finished, remove the bags and place them in the appropriate compartments.
- G. When using the instruments, before opening the bag, complete the patient's form (if one does not already exist)
- H. Remove the previously attached label and stick it onto the patient's form.
- I. At this stage, the tracking system has completed its path. This simple procedure assures us of the tracking of the instruments at any time, it provides us with the sterilization date of each and every instrument and the name of the patient on whom each of the sterile instruments was used. It also enables checking of the sterility expiry date.

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